Diagnosticum Rt.

ARCHITECT

KIT:

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Reaction definition

Reaction mode: End UP
Primary wavelength: **548**
Secondary wavelength: **700**
Last read required: 33
Absorbance range: -0.1000-3.2000

Main read time: 32– 33
Flex read time: 0 - 0
Color correction read time: 0 - 0
Blank read time: **23-24**

Sample blank type: **SELF**
Blank assay:

Reagent/Sample

Reagent: KREAE
R1 reagent volume: 122
R1 water volume: 0
R1 dispense mode: Type 0
Diluent name: Saline

R2 reagent volume: 40
R2 water volume: 0
R2 dispense mode: Type 2
Diluent dispense mode: Type 0

Dilution name	Sample volume	Diluent sample volume	Diluent volume	Water volume	Dilution factor
No Dil	5.5				1:1.00
red 4x	35.0	5,5	165		1 : 4,86

Validity checks

Reaction check type: None
Read time A range:
Calculation limit:

Read time B range:
Minimum absorbance:
Rate linearity %:

Maximum absorbance variation

Calibration parameters

Calibration method: Linear
Use cal factor from: Factor:
Full interval hours:0 Adjustment interval hours: 0
Adjustment type: None Adjustment level:Blank
Expected cal factor: 0 Default ordering type: Full
Expected cal factor tolerance %: **10** Blank absorbance range: 0.0000-0.0000
Span:BLANK Span absorbance range: 0.00-0.00
Maximum curve fit: 0.00
Calibrator set name: Lyo KAL Replicates: 2

Cal level	Concentration	Sample volume	Diluent sample volume	Diluent volume	Water volume
Lyo KAL1	0	5.5			
Lyo KAL2	*	5.5			

* Zadáme deklarovanou hodnotu použitého kalibrátoru (deklarace Enz,)

Smart wash

Component	Reagent/Assay	Wash	Volume	Replicates	Wash protocol
Cuvette	TGL	10%Detergent B	345		

Results parameters:

Linearity range: **0 – 1770**

Flag range specifications:

Interpretation parameters:**Results units**

Results concentration units: umol/l
Results decimal places: 1

Correlation factor: 1.0000
Intercept: 0.0000